

## ADVANTAGES

- **Less time and work for Doctor** — When patient is ready for final denture you already have the trial complete in wax. Any changes or corrections can very easily be done in the wax stage if necessary.
- **Less Laboratory Cost** — Your cost for the two dentures is much less than if you made a treatment denture first and then made the final denture later (no trays, bite blocks, set up, etc.)
- **Happy Patient** — Your patient will be happy knowing that they never have to go without their teeth. Also, because the dentures are duplicates of each other, their appearance never changes going from their treatment denture to their final denture.



### KIESS KRAFT DENTAL LAB

6601 South 118th Street  
Omaha, NE 68137

Phone: 402-391-8424  
Fax: 402-391-5701

Visit our website at: [www.kiesskraft.com](http://www.kiesskraft.com)  
Email: [kkdl@kiesskraft.com](mailto:kkdl@kiesskraft.com)

KIESS KRAFT DENTAL LAB

## TWIN DENTURE TECHNIQUE

*An improvement in  
immediate dentures!*

1-800-553-9522

## YOU'LL LIKE THESE LOOK ALIKES...

...and that will go double for your patients.

There's a lot to like about the Twin Denture concept for both dentist and patient. When you prescribe the Twin Denture service, you will receive two look alike dentures, both made to your exact specifications. They will be identical in esthetics, fit and appearance, but not in material.

One denture will be the acrylic immediate to be inserted at the time of extraction. The other is the twin waxup try-in to be set aside for later use.

The patient wears the immediate during the healing process, confident in the knowledge that the final denture will look the same; or the waxup can be changed or improved to satisfy you and the patient. This eliminates the anxiety and trauma often experienced by patients due to fear of the unknown.

During the healing process the immediate can be relined and reworked as necessary for the patient's comfort and to accommodate the recession of bone and tissue. This adjusting and relining process will more than likely affect the appearance and occlusion of the immediate, making it a less than perfect pattern from which to make the final denture.

This is where the twin demonstrates its value. It is still the exact image of the immediate as it looked when first inserted. The waxup twin is used to take final impressions, and make any change in the esthetics or the bite and function for the final appliance. The patient continues to wear the immediate while the final denture is made using the twin as the model. The patient is happy because he/she is never without teeth. He/she comes in wearing the immediate and departs wearing the final and carrying the immediate which can now be placed in reserve to use in case the final denture is lost or damaged.

Our unique processing system makes the Twin Denture available for only a few dollars more than the traditional one-denture immediate procedure, and a later relin.

Call us with any questions you may have about this service.

## FIRST APPOINTMENT

### Doctor:

1. Takes impressions and centric bite registration.
2. Sends impressions to Kiess Kraft

### Lab:

1. Returns temporary denture to doctor with exact set up or corrections per prescription.
2. Returns baseplate with teeth set up in wax. It is the exact duplicate of the temporary denture.
3. Surgeon's guide (if requested).

## SECOND APPOINTMENT

### Doctor:

1. Extractions & etc.
2. Inserts temporary denture for healing process.
3. Keeps baseplate with teeth set for trial on hold. This will be used for third appointment.

## THIRD APPOINTMENT

(after shrinkage & healing has taken place)

### Doctor:

1. Uses baseplate with trial set-up as custom tray, and takes corrective impression inside baseplate.
2. Sends impression to Kiess Kraft.

### Lab:

1. Pours stone model and processes denture.
2. Returns finished denture to Doctor.

## FOURTH APPOINTMENT

### Doctor:

1. Inserts finished or final denture.

(Patient now has two dentures — final denture and treatment denture for a spare)

### KIESS KRAFT DENTAL LAB

6601 South 118th Street  
Omaha, NE 68137

Phone: 402-391-8424  
Fax: 402-391-5701

Visit our website at: [www.kiesskraft.com](http://www.kiesskraft.com)  
Email: [kkdl@kiesskraft.com](mailto:kkdl@kiesskraft.com)